

# Specimen Collection and Submission Instructions Carbapenem-resistant *Pseudomonas* Carbapenemase (Version 1) Page 1 of 2

Specimen Type	Collection Time	Collection Frequency	Collection Procedures	Transport Media	Shipping & Handling (S&H)*
					<b>Transport device:</b> Choc, HIA, BHI slant or plate. Note: Plate may be used <i>only</i> if transported by courier.
Non-mucoid <i>Pseudomonas aeruginosa</i> isolate resistant to any carbapenem:  MIC of ≥8 µg/ml for any carbapenem	NA	NA	NA	Choc, HIA, BHI slant or plate.  Note: Plate may be used only if transported by courier.	<ul> <li>Rejection Criteria:         <ul> <li>Contaminated media.</li> <li>Does not meet CR-Pseudomonas testing criteria.</li> <li>Broken transport device.</li> <li>Leaking specimen.</li> <li>Requisition form/isolate discrepancies.</li> </ul> </li> <li>Transport: Ship at ambient temperature.         <ul> <li>Ship as Category B.</li> </ul> </li> <li>Storage: Store at ambient temperature.</li> </ul>



# Specimen Collection and Submission Instructions Carbapenem-resistant *Pseudomonas* Carbapenemase (Version 1) Page 2 of 2

## **General Rejection Criteria (for additional details, see S&H)**

• Failure to follow specific S&H requirements.

#### NOTE:

Specimens will not be processed until ALL of the following information are provided on requisition form:

- Patient name, patient's county of residence, and second identifier.
- Two matching identifiers on both the tube and the laboratory requisition form.
- Specimen type, date of collection, and test requested.
- Submitter name, address, and telephone/FAX numbers.
- Reason for testing at WAPHL.

### **Collection kit availability**

Not available.

### **Comments**

None.