

Specimen Collection and Submission Instructions Expanded Antimicrobial Susceptibility Testing for Hard to Treat Infections (ExAST) (Version 1)

Specimen Type	Collection Time	Collection Frequency	Collection Procedures	Transport Media	Shipping & Handling (S&H)*
<ul> <li>Enterobacteriaceae isolates that:</li> <li>Test non-susceptible to all beta-lactams, including either ceftazidime/avibactam or meropenem/vaborbactam (these isolates may be metallo-beta-lactamase-producing isolates with few effective treatment options)</li> <li>Possess metallo-beta-lactamase genes (NDM, VIM, or IMP) confirmed by a molecular test</li> </ul>	NA	NA	NA	Choc, HIA, BHI slant or plate. Note: Plate may be used <i>only</i> if transported by courier.	<ul> <li>Transport device: Choc, HIA, BHI slant or plate. Note: Plate may be used only if transported by courier.</li> <li>Rejection Criteria: <ul> <li>Non-AR Lab Network approved samples.</li> <li>Contaminated media.</li> <li>Does not meet ExAST testing criteria.</li> <li>Broken transport device.</li> <li>Leaking specimen.</li> <li>Requisition form/isolate discrepancies.</li> </ul> </li> <li>Transport: Ship at ambient temperature. Ship as Category B.</li> <li>Storage: Store at ambient temperature.</li> </ul>



Specimen Collection and Submission Instructions Expanded Antimicrobial Susceptibility Testing for Hard to Treat Infections (ExAST) (Version 1)

## General Rejection Criteria (for additional details, see S&H)

• Failure to follow specific S&H requirements.

NOTE:

Specimens will not be processed until ALL of the following information are provided on requisition form:

- Patient name, patient's county of residence, and second identifier.
- Two matching identifiers on both the tube and the laboratory requisition form.
- Specimen type, date of collection, and test requested.
- Submitter name, address, and telephone/FAX numbers.
- Reason for testing at WAPHL.

## **Collection kit availability**

Not available.

## **Comments**

None.