

## Specimen Collection and Submission Instructions

### *Listeria monocytogenes* (Environmental) (Version 1)

Page 1 of 2

Specimen Type	Collection Time	Collection Frequency	Collection Procedures	Transport Media	Shipping & Handling (S&H)**
Food	ASAP	NA	<ul style="list-style-type: none"> <li>• If possible send in original packaging or container.</li> <li>• Sterile/Aseptic technique is necessary for handling and packaging food items.</li> <li>• Items must be individually bagged in sterile, leak-proof containers.</li> <li>• Amount: 50-100 grams (1.7-3.5 ounces) of specimen is required.</li> </ul>	NA	<ul style="list-style-type: none"> <li>• <b>Transport device:</b> Sterile, leak-proof bag or container.</li> <li>• <b>Rejection Criteria:</b> <ul style="list-style-type: none"> <li>○ Testing not approved by CDE.</li> <li>○ Leaking or inappropriately packaged specimen.</li> <li>○ Improper storage or shipping conditions.</li> </ul> </li> <li>• <b>Transport:</b> See storage requirements.</li> <li>• <b>Storage:</b> Keep refrigerated foods at 0°-4°C. Frozen foods should be kept frozen. Dry/canned goods may be stored at room temperature.</li> </ul>
Environmental Swab	NA	NA	<ul style="list-style-type: none"> <li>• Sterile/Aseptic technique is necessary for handling and packaging swabs.</li> <li>• Swabs must be individually bagged in sterile, leak-proof containers.</li> </ul>	NA	<ul style="list-style-type: none"> <li>• <b>Transport device:</b> Sterile, leak-proof container.</li> <li>• <b>Rejection Criteria:</b> <ul style="list-style-type: none"> <li>○ Testing not approved by CDE.</li> <li>○ Leaking or inappropriately packaged specimen.</li> <li>○ Improper storage or shipping conditions.</li> </ul> </li> <li>• <b>Transport:</b> Cold (Not frozen).</li> <li>• <b>Storage:</b> Refrigerate.</li> </ul>

**General Rejection Criteria (for additional details, see S&H)**

- Unaccepted specimen type.
- Leaky specimen.
- Failure to follow specific S&H requirements.

**Collection kit availability**

Not available.

**Comments**

- If necessary, please contact laboratory before collection or shipping for additional assistance.
- For Environmental Swabs, commercial swab/sponge collection kits may be acceptable. Please call lab before collection to verify.