

# Specimen Collection and Submission Instructions

## Anaerobic Bacterial Culture (Version 1)

Page 1 of 2

Specimen Type	Collection Time	Collection Frequency	Collection Procedures	Transport Media	Shipping & Handling (S&H)**
Clinical specimens suspected of anaerobic infection	NA	NA	Please follow appropriate anaerobic specimen collection protocols.	Anaerobic transport media	<ul style="list-style-type: none"> <li>• <b>Transport device:</b> As appropriate.</li> <li>• <b>Rejection Criteria:</b> <ul style="list-style-type: none"> <li>○ Inappropriate transport media.</li> <li>○ Expired transport media.</li> <li>○ Non-viable specimen.</li> <li>○ Broken transport device.</li> <li>○ Leaking specimen.</li> </ul> </li> <li>• <b>Transport:</b> Ship ASAP on ice packs. Ship as Category B.</li> <li>• <b>Storage:</b> Store at 2 – 8°C.</li> </ul>

**General Rejection Criteria (for additional details, see S&H)**

- Failure to follow specific S&H requirements.

NOTE:

Specimens will not be processed until ALL of the following information are provided on requisition form:

- Patient name, patient's county of residence, and second identifier.
- Two matching identifiers on both the tube and the laboratory requisition form.
- Specimen type, date of collection, and test requested.
- Submitter name, address, and telephone/FAX numbers.
- Reason for testing at WAPHL.

**Collection kit availability**

Not available

**Comments**